

## SCALPELS

## #3 Standard



12 cm

No. 10003-12

## #3 Secure


A safety lock holds the blade securely, and allows simple and safe blade attachment and removal.




12 cm

No. 10003-13

- 1 Slide blade onto front pin of the handle.

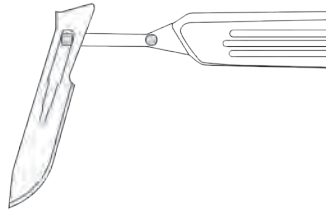

- 2 Then slide pin to front slot of blade. Rotate blade clockwise and secure blade to back pin on handle.

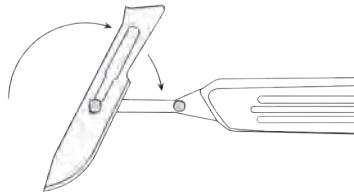

- 3 Scalpel is ready to use.



## Blades for #3 and #7 Scalpel Handles

The alloy used has a high carbon content which provides a longer lasting edge. Blades come individually wrapped. **Non-sterile.**



Box. of 100 blades

No. 10010-00



Box. of 100 blades

No. 10011-00



Box. of 100 blades

No. 10012-00



Box. of 100 blades

No. 10015-00